CT HeadFix

REF Products
1105 CT HeadFix

Sistema de posicionamiento para la fijación de la cabeza en la base TC
Sistema de posicionamento para fixação da cabeça no aparelho de TC
Placering af anordningen til fiksering af hovedet i CT-hovedholderen
Apuväline potilaan pään asemoimiseen CT päätukeen
Pasientposisjoneringshjelp for hodefiksering i CT-hodeholderen
Positioneringshjälp för huvudfixering i CT-huvudhållaren
Positionerings hulpstuk ten behoeve van het fixeren van het hoofd wanneer rustend in het CT hoofd steun
ตัวช่วยปรับตำแหน่งการยึดศีรษะในหัวรองรับศีรษะสำหรับทำ CT
Приспособление для фиксации головы в головке корпуса КТ.
Uloženie pacjenta w urządzeniu TK
Staðsetningaraðstoð fyrir höfuðfesti í CT höfuðhaldara
CT Head Holder CT [헤드 홀더] 내 머리 고정을 위한 포지셔닝 보조 장치
用於在 CT 頭部支撐器中固定頭部的定位輔助
Bilgisayarlı tomografi kafa tutucu için kafa pozisyonlama seti
Universaalne positsioneerimise abivahend MR ja CT uuringutel
Pomoc na polohu na fixovani hlavy v CT v nádobě, pro hlavu.

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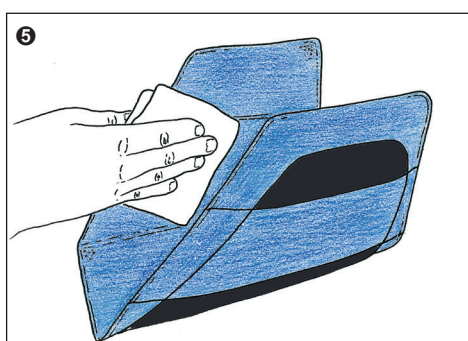
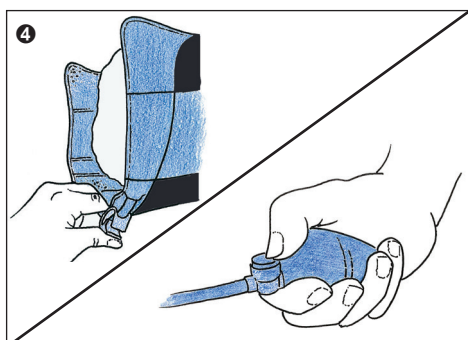
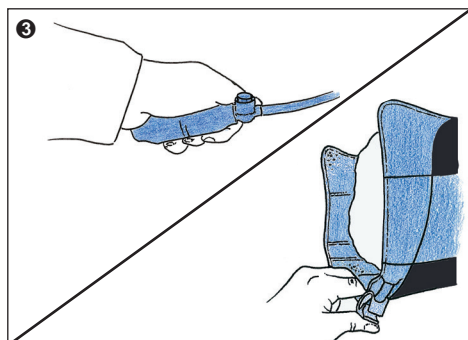
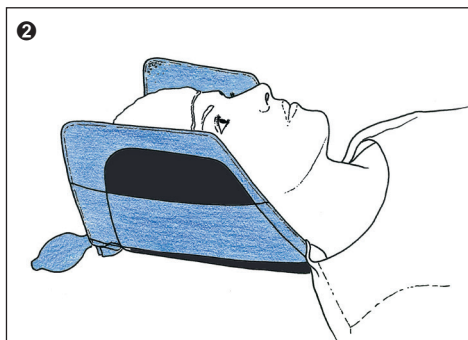
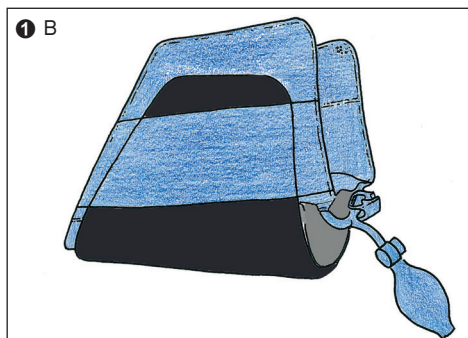
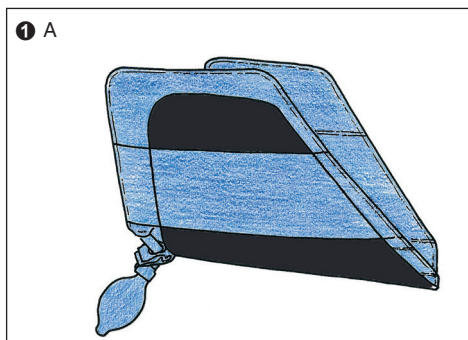
Positioning aid for head fixation in the CT head holder

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Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

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1 **Version A** (without inclination wedge): Put the cleaned CT HeadFix into the head holder and secure it using the side straps.

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1 **Version B** (with inclination wedge): First put the inclination wedge into the head holder. Then put the cleaned CT HeadFix into the head holder and secure it with the side straps.

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2 The head is placed into the head holder and positioned to prepare for measurement. A hygiene barrier (e.g. a head cover) must always be used for hygienic reasons and to protect the patient.

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3 Inflate the CT HeadFix until the required immobilization is achieved. Do not over-inflate. The pressure applied must be adjusted individually to the patient's condition by qualified personnel. If required, close the hose clamp to prevent the exchange of air between the two air chambers.

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4 Once the examination is complete (see the device manufacturer's instructions for use), undo the hose clamp, if necessary and deflate the CT HeadFix by pressing the valve on the hand pump.

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5 Dispose of used hygiene barriers and wipe the CT HeadFix with any regular disinfectant. Do not immerse the CT HeadFix in cleaning fluid.

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Contraindications: Do not use the product with children below the age of 3. Do not use the product for purposes other than those described.

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 Non sterile.

 Nicht steril.

 Do not sterilize. A hygiene barrier (mechanical barrier) must always be used. Do not inflate with compressed air. The pressure applied must be adjusted individually to the patient's condition by qualified personnel. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare professional (Rx only).

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