Multipad

REF Products

1012 Multipad Standard
1022 Multipad Plus
1071 Multipad Slim
1104 Multipad Ear
1108 Multipad Bendy

Sistema de posicionamiento de pacientes de utilización universal para RM y TC
Sistema de posicionamento do doente, universalmente utilizável em imagiologia por RM e TC
Universelt anvendeligt patientpositioneringssystem til MR- og CT-scanning
Yleiskäyttöinen potilaan asemointijärjestelmä MR- ja CT-kuvantamiseksi
Universelt pasientposisjoneringssystem for MR- og CT-bilder
Universellt användbart patientpositioneringssystem för magnetresonans- och datortomografi
Universeel toepasbaar patiëntpositionerend systeem ten behoeve van MR- en CT-opnamen
ระบบกำหนดตำแหน่งผู้ป่วยเอกซเรย์รังสีสำหรับ MR และการจับภาพ CT
Универсальная система для позиционирования пациента при МРТ и КТ
Uniwersalny system do stabilizacji pacjenta do diagnostyki obrazowej MR i CT
Alhliða skorðunarbúnaður fyrir sjúklinga við segulómmyndun og tölvusneiðmyndun
CT & MRI 펠텍 공기 환자 보조 장치
MR 及 CT 通用病患固定裝置
CT ve CT görüntüleme için evrensel pozisyonlama seti
Universaalne positsioneerimise abivahend MR ja CT uuringutel
Univerzální polohovací pomůcka pro MR a CT vyšetření

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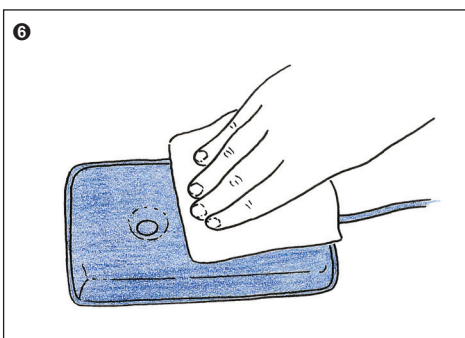
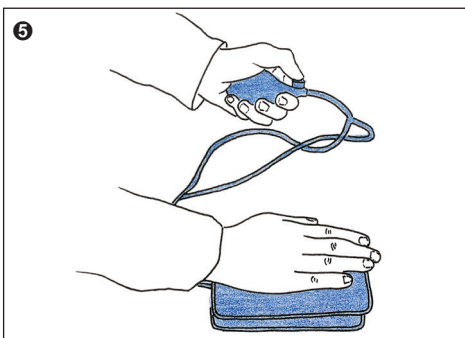
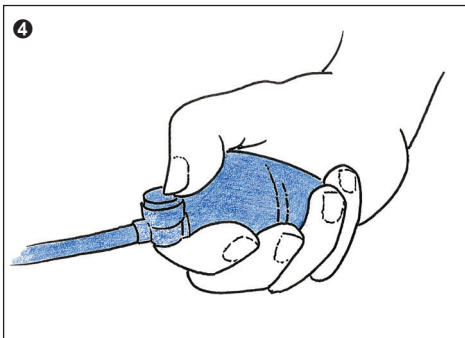
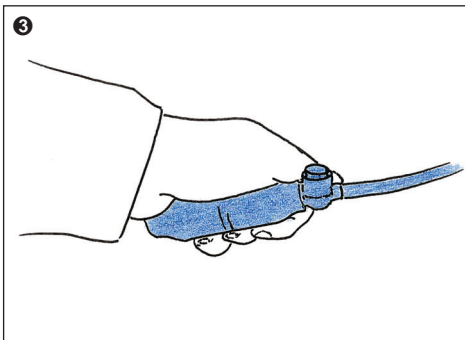
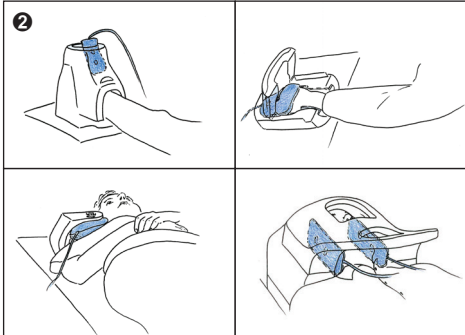
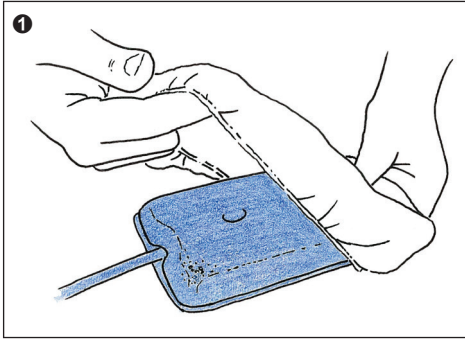
Universal patient positioning aid for MR and CT imaging

Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

1 Before using on a patient, cover the disinfected Multipad with a single-use sanitary cover (Cover Multipad).

Note: Using the cover is essential for sanitary reasons as well as to protect the positioning system.

2 Move the patient into the correct position in preparation for the examination. If necessary, position the Multipad between the part of the body that needs stabilizing and the MR coil or CT support.

Note: The polystyrene pearls should face the patient.

3 Pump up the Multipad until the required degree of support has been attained. Do not over-pump. The patient should find the pressure pleasant. Position the pump and tube to avoid them from getting caught when the patient is moved into the scanner. Position the pump far away from the field of view.

4 Once the examination is complete (see instructions provided by the device manufacturer), deflate the Multipad by pressing the valve on the hand pump. Remove the Multipad from the patient.

5 Dispose of the single-use sanitary cover (Cover Multipad) and squeeze the remaining air out of the Multipad by hand.

6 Clean the Multipad using any readily available hospital disinfectant. The disinfectant should be applied as directed but thoroughly wiped off after application. Note: Do not immerse the Multipad in cleaning fluid. Optimal functionality guaranteed until red indication line appears on the Multipad.

Contraindications: Do not use the product with children below the age of 3. Do not use the product for purposes other than those described.

MR MR safe

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! Do not sterilize. Do not allow direct contact with the skin. Use single-use sanitary cover. Do not use in case of acute open wounds. Do not inflate with compressed air. Position the pump far away from the field of view. Caution: Federal law restricts this device to sale by or on the order of a licensed health-care professional (Rx only).

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